

## How to change batteries for ZOLL AED Plus®:



- To remove the battery cover, insert the Defibshop® keyring as shown in the picture or a small tool (e.g., flat-head screwdriver or coin) into the two slots on the rear of the unit to depress the latches, then insert the tool into the groove at the bottom to lift the cover.
- Remove all batteries at once and dispose of batteries thoughtfully.
- Place new batteries in the battery bank, ensuring all batteries are securely seated.
- After placing the first five batteries, the audio prompt begins.
- After installing all new batteries for **up to 15 seconds**, the audio prompt “IF NEW BATTERIES PRESS BUTTON” reminds you to press the small **blue reset button** inside the battery compartment.
- **Press and hold** the button for **at least 3 seconds**.
- The unit will display a **Green Tick**, as shown below, in the status indicator window on the front of the unit.



For more information, don't hesitate to contact our team at **1300 729 575**.